

EQUI PATCH



RAPAIR MEDICAL DEVICES



EquiPatch should be applied within 24 hours of the wound occurring for optimal results.

EquiPatch is a reusable silicone dressing clinically proven to reduce the healing time of all types of cuts and lacerations from weeks to days, whilst minimising scarring and proud flesh.

EquiPatch was developed by Dr Rosemary Craig, MBBS, FRACGP, a rural doctor, horse owner and winner of numerous National and State RDA Dressage Titles.

HOW DOES EQUIPATCH WORK?

Skin contains three main layers, the outside layer is the epidermis, below that is the dermis and below the dermis is subcutaneous fat. The dermis is predominately made up of a matrix of collagen and other molecules. In healthy skin new molecules align and attach to the existing matrix. When skin is damaged molecules flow into the wound but are unable to align properly. This contributes to slow healing rates and scar formation.

EquiPatch contains fine structures that imitate physical properties of intact skin. These structures project electromagnetic guidelines into the wound so the collagen and other molecules can align as if the skin is still intact. This dramatically reduces healing times and produces a minimal scar.

CLINICAL EVIDENCE:

In a clinical trial of EquiPatch stitches were removed from excision wounds within 24 hours. Normally these stitches would need to stay in for at least 10 to 14 days.

A panel of clinicians that reviewed wounds from these trials rated the one-day old wounds treated with EquiPatch as more than eight days old.

Case studies on horses have demonstrated rapid reformation of the dermis with wounds healing within a few weeks with no visible scars.

Phone: 1800 727 247
Email: info@equipatch.com.au



WWW.EQUIPATCH.COM.AU

MANUFACTURED BY RAPAIR MEDICAL DEVICES



DIRECTIONS FOR USE

1. Clean the wound.
2. Apply EquiPatch by itself directly over the wound with the white side closest to the skin and the yellow stripes aligned along the length of the wound.
3. Secure EquiPatch with padding and a cohesive bandage.
4. EquiPatch can be removed after 24 hours or left for up to 3 days.
5. Treat the wound as required until fully healed.

COMPOSITION:

Soft Silicone containing fine ferro-magnetic structures.

STORAGE:

Store EquiPatch flat in its packaging away from direct sunlight, at ambient temperature (5-30°C) and humidity.

CLEANING:

Rinse EquiPatch with clean water, dry and store flat in the packaging for reuse.

USING EQUIPATCH

EquiPatch can be used anywhere on the horses body.

EquiPatch can be cut to any shape or size.

EquiPatch can be used over stitches or staples.

Multiple patches can be used on the same wound as long as the stripes are orientated the same way and the edges of the patches do not overlap.



Standard: 10cm x 5cm



Long: 16.5cm x 5cm



Wide: 10cm x 10cm

Custom sizes available by special order

Phone: 1800 727 247
Email: info@equipatch.com.au



WWW.EQUIPATCH.COM.AU

MANUFACTURED BY RAPAIR MEDICAL DEVICES